




BD PureHub™ Disinfecting Cap

Usage guidelines

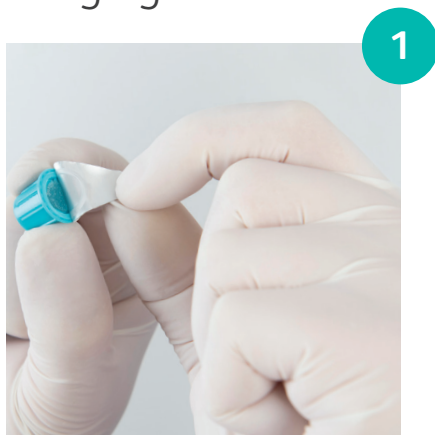

Peel the foil from the BD PureHub™ disinfecting cap.



Apply the cap to the needle-free luer connector by **pushing and twisting**.



The BD PureHub disinfecting cap must remain on the needle-free luer connector for a minimum of one (1) minute and may remain on for up to seven (7) days.

Ordering information

Material number	Description	Packaging
306604	BD PureHub disinfecting cap	Singles

Provides a >4 log (99.99%) reduction of bacteria within 1 minute.*
Discard cap after removal. Single use only.

* Demonstrated reduction on the most common causative agents of catheter-related bloodstream infection including *Staphylococcus aureus*, *Staphylococcus epidermidis*, *Pseudomonas aeruginosa*, *Escherichia coli*, *Candida glabrata* and *Candida albicans*.

Threaded plastic cap is not made with natural rubber latex or plasticiser diethylhexyl phthalate. Active ingredient: 70% isopropyl alcohol.

Please read labelling at all times.
Always perform disinfection techniques as prescribed by hospital protocol.

BD Switzerland Sàrl. Terre Bonne Park – A4. Route de Crassier 17. 1262 Eysins, Switzerland

bd.com

© 2018 BD. BD, the BD Logo and BD PureHub are trademarks of Becton, Dickinson and Company.
0000CF03429 Issue 1

